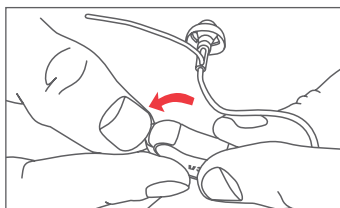
Quick Reference Guide: Getting Started

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

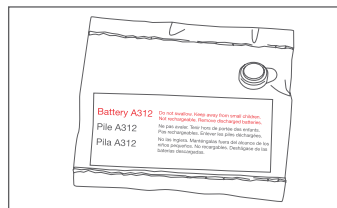
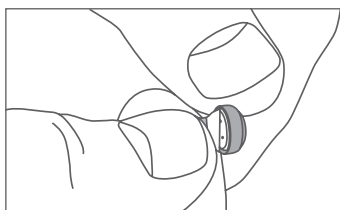
4

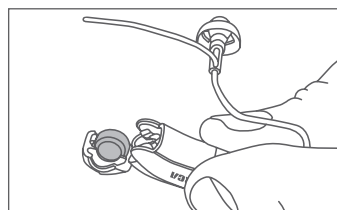5

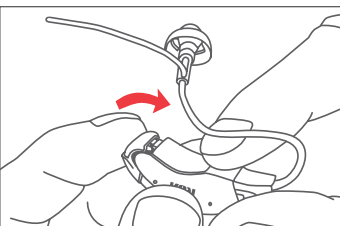IMPORTANT: Once you close the battery door, the personal sound amplifier is on and ready to use. The earpiece stays on until the battery runs out of power or you open the battery door again. If you're not going to use the earpiece for a while, open the battery door to conserve battery power.

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1 Open the battery door on the back of the earpiece.

2 Find the battery bag (inside the storage case). Remove the battery from the bag.

3 Peel the protective tab off the back of the battery.

4 Place the battery into the battery door.

5 Close the battery door completely.

NOTE: Normal battery life for the earpiece is 7 full days of use.

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

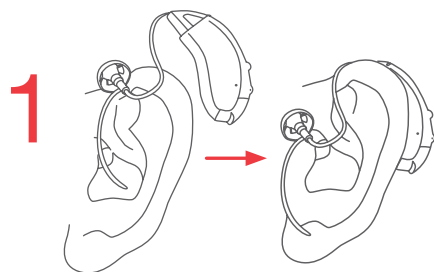
Using / Utilisation / Modo de uso

Note: The earpiece comes with the thin right-ear tube and medium-sized dome attached. This package includes other tubes and domes, including a wide tube that provides more volume. See the User Guide for instructions for changing.

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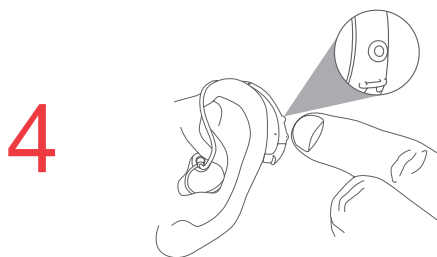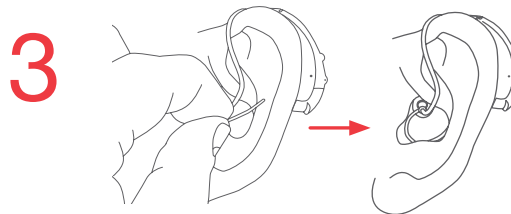1 Put the earpiece body behind your ear.

2 Insert the dome in your ear gently and wiggle it into the ear canal.

3 Fold the tail into the bowl of your ear.

4 Press the volume button to go to setting 2. Press again to go to level 3. Press a third time to go back to level 1.

For more information on using and taking care of your earpiece and charger, see the User Guide.

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